- EN 1. Probe, 2. Display, 3. Battery Cover, 4. ON/OFF/START Button
- FI 1. Mittariin, 2. LCD, 3. Pariston suojus, 4. On/Off-painike
- NO 1. Sondet, 2. LCD-skjerm, 3. Batterihette, 4. På/Ap-knapp
- SU 1. Spets, 2. LCD, 3. Batterilucka, 4. På/Va- knapp

**EN English**

**Introduction**

The Digital Thermometer provides quick, highly accurate reading over the body temperature range. Please read the instruction completely first. If thermometers are used by the children, the supervision of the adult is compulsory. The quality of the device has been verified and conforms to the provisions of the EC council directive 93/42/EEC. (Medical Device Directive) Annex I essential requirements and applied harmonized standards: ISO 80601-2-56: 2017+A1: 2018 Medical electrical equipment -Part 2-56: Particular requirements for basic safety and essential performance of clinical thermometers for body temperature measurement.

Attention: Consult the accompanying documents. Please read this manual carefully before use. Please be sure to keep this manual.

**Battery Replacement**

Place a new 1.5V D.C. button size battery type SR41 or LR41 or equivalent in the chamber with positive side faced up and negative side faced down.

**Display on the LCD**

M: Last measured temperature.

When the appears on the bottom right corner of the LCD, the battery is exhausted and needs replacement.

**Troubleshooting**

Error message	Problem	Solution
	The system is not functioning properly	Unload the battery, wait for 1 mins and repower it. If the message reappears, contact the retailer for service.
<b>H</b> <b>(bi-bi...bi)x10</b>	Temperature taken is higher than 42.9°C (109.9°F)	Take a new temperature via close contact and sufficient rest.
	Temperature taken is lower than 32°C (90.0°F)	Take a new temperature via close contact and sufficient rest.
	Dead battery: Battery icon appears, can't be measurable.	Suggest to replace the battery.

**Cleaning and Disinfection**

- 1. Clean the unit by wiping it with a dry cloth and disinfect the probe with ethyl alcohol.
- 2. Don't let the unit contact any chemical thinner.
- 3. Do not soak the display in water.

**Caution**

- Do not bend, drop or twist the thermometer. It is not shockproof.
- Do not store the unit under direct sunlight, at a high temperature, in high humidity or dust.
- Do not disassemble. See BATTERY REPLACEMENT to replace battery.
- Avoid strong electromagnetic interference such as microwave ovens and cell phones.
- Keep battery away from children. Harmful is swallowed.
- Battery should not be charged or placed into extreme heat as it may explode.
- Remove battery from the thermometer when not in operation for a long time.
- If the unit is stored in an extreme environment before measurement, please make sure it has been firstly adapted in the room temperature, so that the unit can be measured normally.
- Measurement results are for reference only. Contact your physician if you have or suspect any medical problems. Do not change your medications without the advice of your physician or healthcare professional.
- This device may not meet its performance specification if stored or used outside temperature and humidity ranges specified in specifications.
- It is recommended the performance should be checked every two years.
- Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.
- High, prolonged fever requires medical attention especially for young children. Please contact your physician.
- For safe reason, during children's temperature measurement, please keep them from crying, walking, talking and any related dangerous activities.
- Usage of the probe cover may result in a 0.1°C(0.2°F) difference from actual temperature.
- To clean the probe before and after using the thermometer to ensure an accurate reading and avoid cross contamination approximately, no matter the probe cover has been installed or not.
- Do not drink hot or cold fluids, exercise, and smoke or perform other activi-

ties prior to a reading. These activities will raise or lower temperature readings when compared to your normal, average temperature.

- Please ensure the close contact between the probe and the armpit of the baby or children.

- Do not service or maintain device while in use.

**Specifications**

Range:	32.0°C–42.9°C (90.0°F–109.9°F)
Accuracy:	±0.1°C/35.5°C–42.0°C (±0.2°F/95.9°F–107.6°F) at standard room temperature 18°C–28°C (64.4°F–82.4°F); ±0.2°C under 35.5°C or over 42.0°C (±0.4°F under 95.9°F or over 107.6°F)
Display:	Liquid crystal display, 3 1/2 digits
Battery:	One 1.5 V D.C. button size battery (SR41 or LR41)
Power consumption:	0.15 milliwatts in measurement mode
Battery life:	Approx. 1500 times operation or 1 year with 1-2 measurement per day including standby mode.
Dimension:	12.3cmx 1.8cm x0.9cm (LxWxH)
Weight:	Approx. 10 grams including battery
Alarm:	Approx. 10 seconds sound signal when peak temperature reached
Environmental for using:	Temperature: 5.0°C–40.0°C (41.0°F–104.0°F), Humidity: 15%–95%RH, 700–1060hPa
Storage and transportation condition :	Temperature: -20.0°C–55.0°C (-4.0°F–131.0°F), Humidity: 15%–95%RH, 700–1060hPa
IP Classification:	IP22, Protection against harmful ingress of water and particulate matter
Safety classification:	Type BF Applied Part

**Electromagnetic Compatibility Information**

This device needs to be installed and put into service in accordance with the information provided in the user manual.

2. WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the TG100, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result.

If higher IMMUNITY TEST LEVELS than those specified in Table 9 are used, the minimum separation distance may be lowered. Lower minimum separation distances shall be calculated using the equation specified in 8.10.

Immunoty	IEC 60601 test level	Compliance level	Electromagnetic environment/guidance
Conducted RF	IEC 61000-4-6	Not applicable	Portable and mobile RF communications equipment should be used no closer than any part of the TG100 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2 \sqrt{P}$ d = 1.2 √P 80MHz to 800 MHz, d = 2.3 √P 800MHz to 2.7 GHz. Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF	IEC 61000-4-3	10 V/m 80 MHz–2.7 GHz 80 % AM at 1 kHz	

NOTE1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

More information on EMC compliance of the device can be obtained from Rossmax website: www.rossmax.com

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