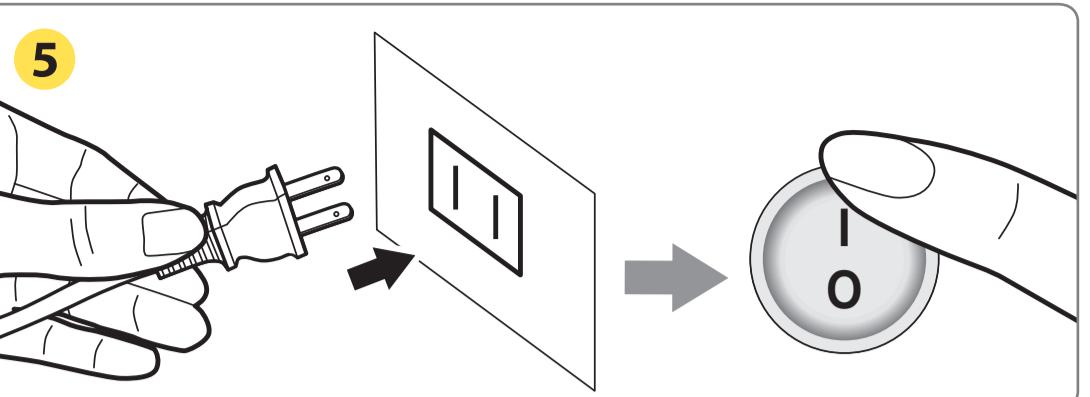
## Setup Before Use

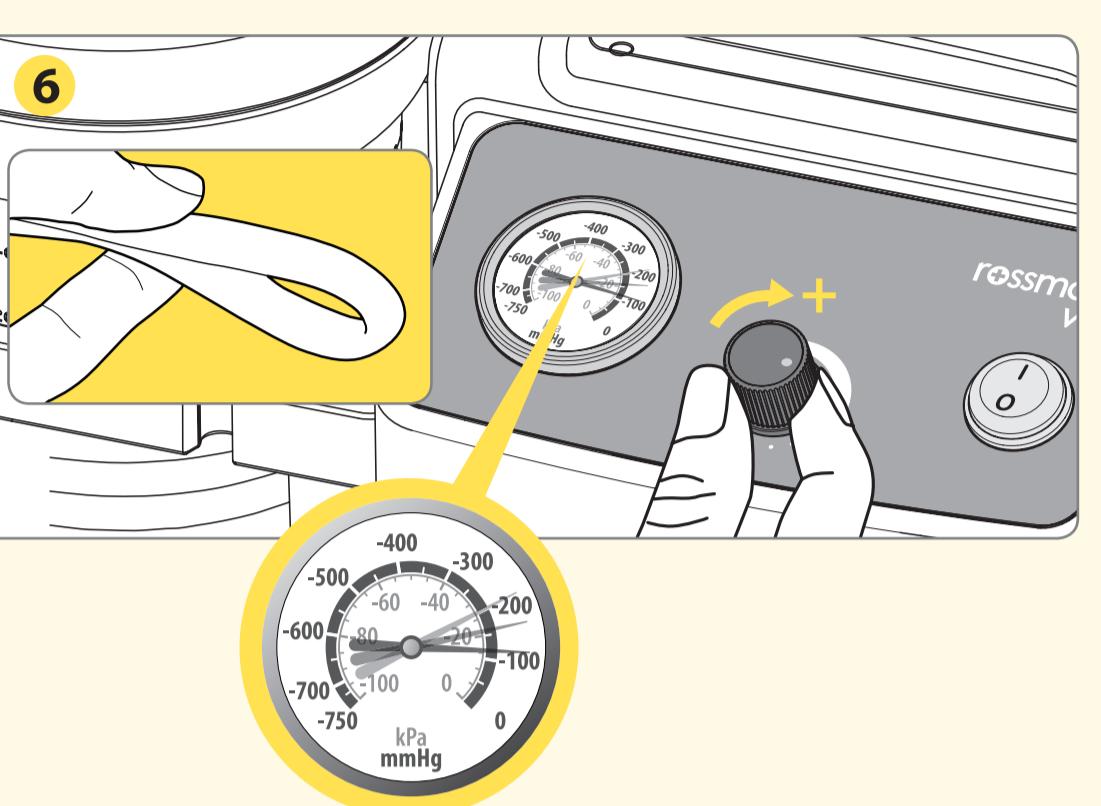
EN Plug the power cord into the socket. Place the switch to the "I" (ON) position.

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
EN Collapse the tubing and check the vacuum gauge to test that the level varies when the vacuum regulator is acted on. Adjust the level of vacuum prescribed by the doctor turning vacuum regulator. Note: Turn the regulator clockwise in the "+4" direction to obtain a higher vacuum; or anticlockwise for a lower vacuum

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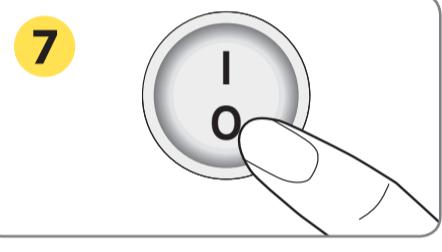
EN Place the switch to the "O" (OFF) position

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## Starting the Suction

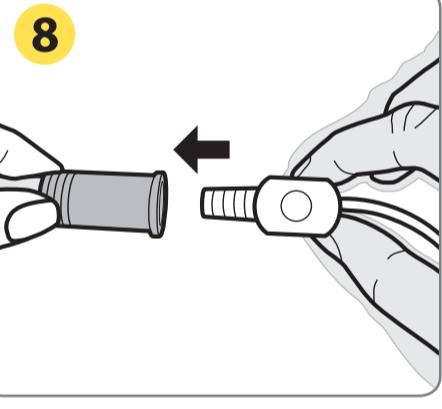EN Connect the suction catheter to the connecting PVC tubing (Ø 1/4", L: 180 cm)

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


**Disassembling Collection Canister**

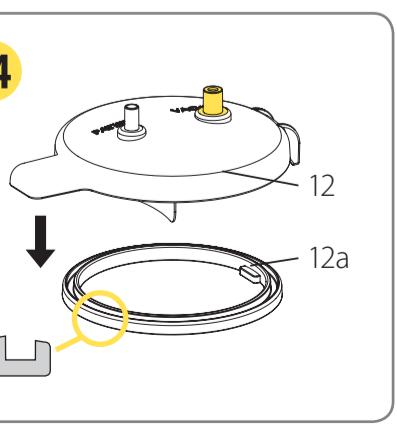
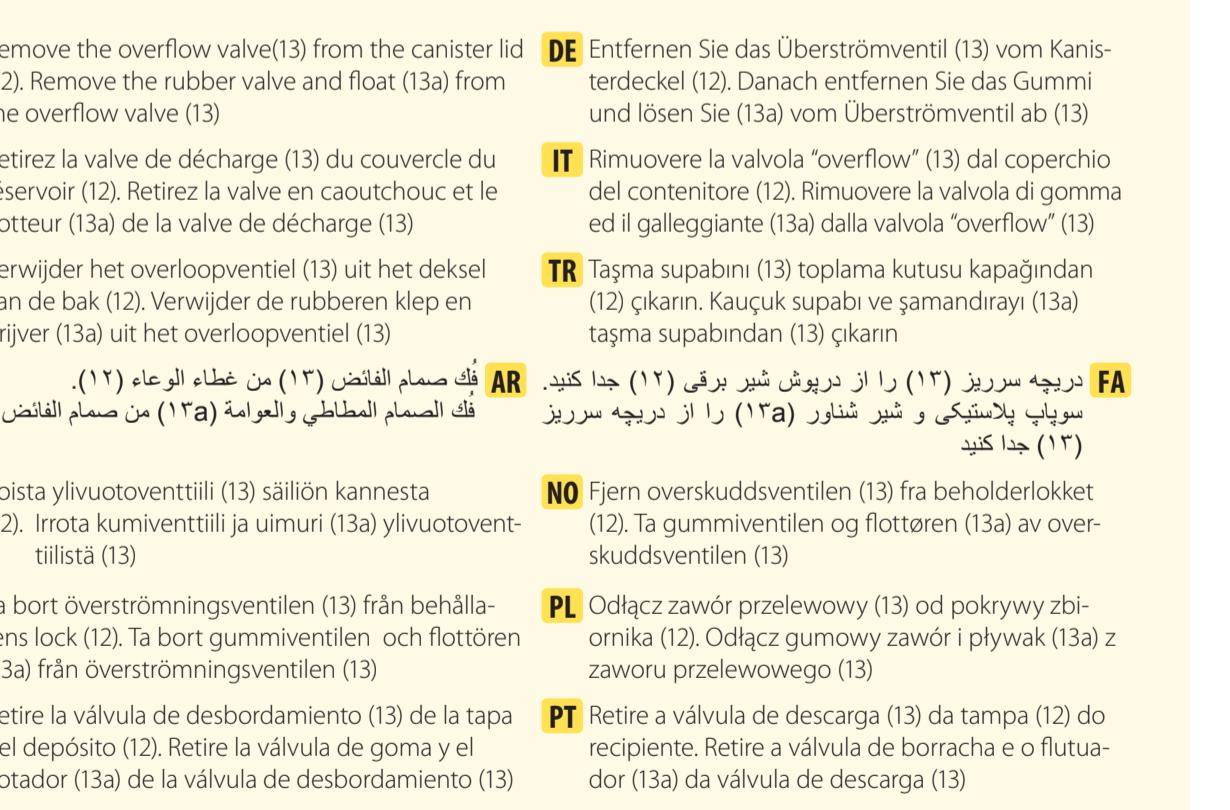
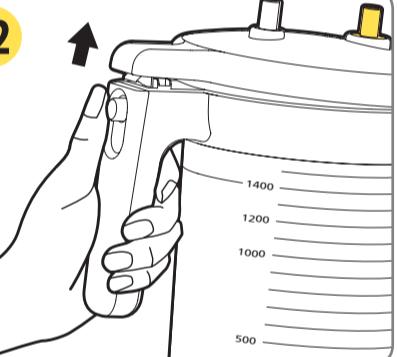
[www.rossmax.com](http://www.rossmax.com)  
**Warranty Card**  
This instrument is covered by a 2 year guarantee from the purchase date. The guarantee is valid only on presentation of the warranty card completed by the dealer confirming purchase date or the receipt. Suction unit accessories are not included. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or noncompliance with the instruction manual. Please contact your local seller/dealer or www.rossmax.com.

**Customer Name:** \_\_\_\_\_  
**Address:** \_\_\_\_\_  
**Telephone:** \_\_\_\_\_ **E-mail address:** \_\_\_\_\_  
**Product Information**  
**Date of purchase:** \_\_\_\_\_  
**Store where purchased:** \_\_\_\_\_

**WARNING:** The symbol on this product means that it's an electronic product and following the European directive 2012/19/EU the electronic products have to be disposed on your local recycling centre for safe treatment.

## Product identification

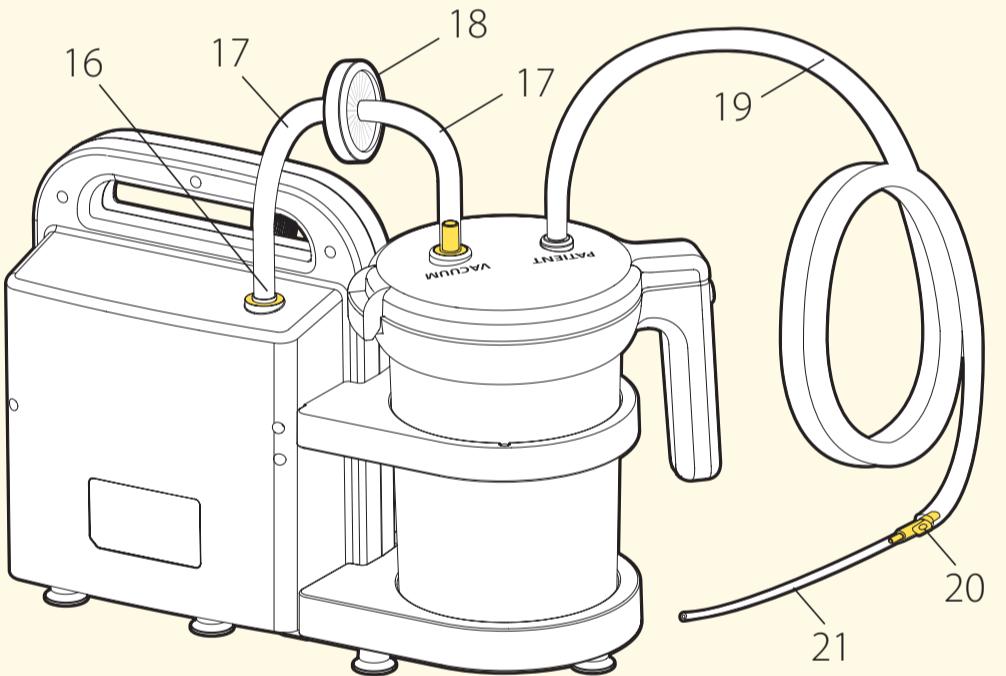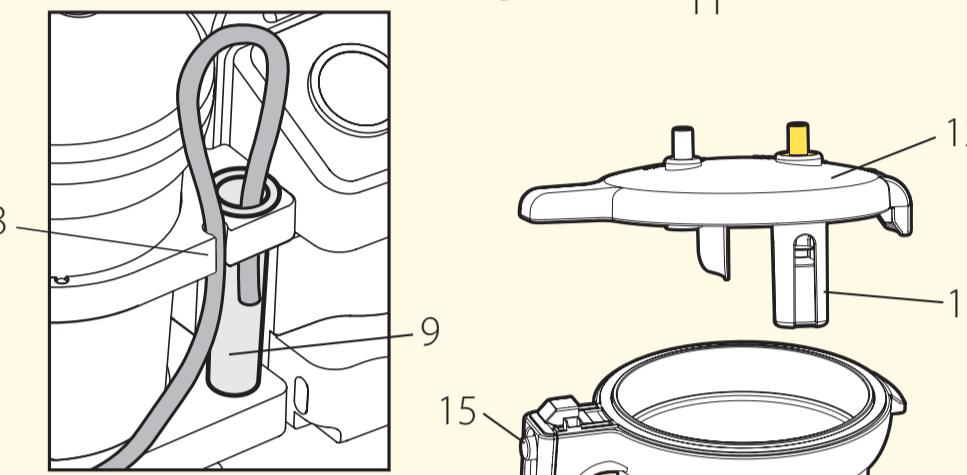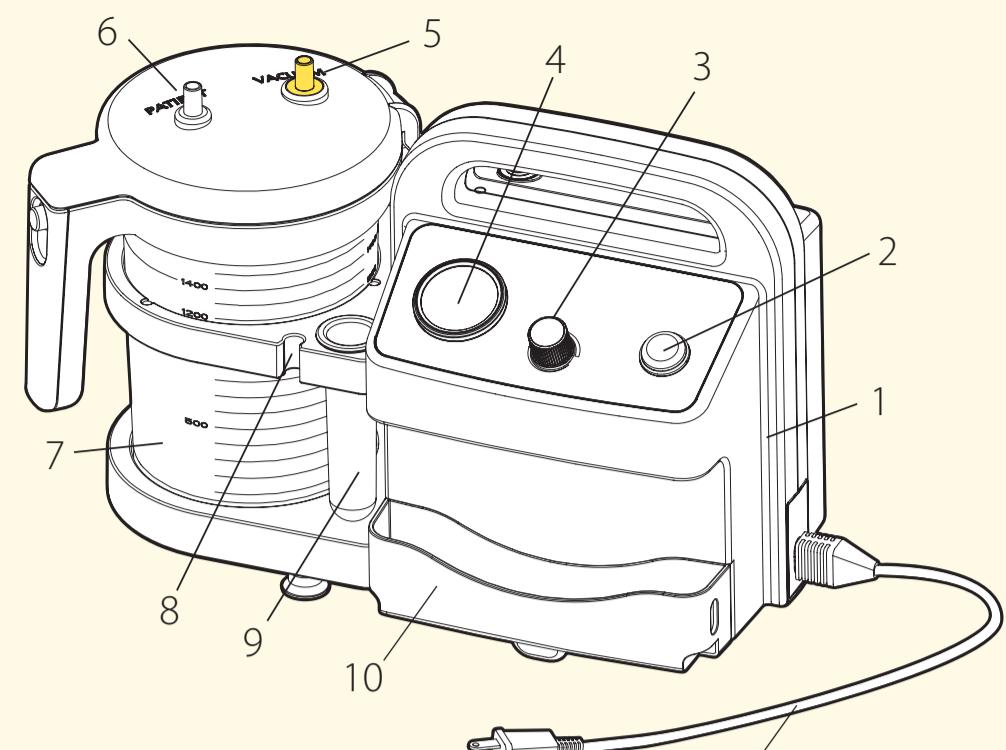**EN**

- Suction unit
- On/Off switch
- Vacuum flow adjustment knob/ vacuum regulator
- Vacuum gauge
- Canister "VACUUM" inlet
- Canister "PATIENT" inlet
- Collection canister (1,400 ml)
- Tubing holder
- Tubing depository
- Accessory compartment
- Power cord
- Canister lid
- Overflow valve
- Canister supporting handle
- Switch button of the canister handle
- Air inlet
- Connecting silicone tubing (Ø8 mm, 14 cm)
- Antibacterial filter
- Connecting PVC tubing (Ø1/4", 180 cm)
- Disposable manual regulator (Single use)
- Disposable suction catheter (Single use)

Note: Disposable manual regulator (20) and Disposable suction catheter (21) are not supplied by Rossmax.

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
**EN** Connect the connecting PVC tubing (Ø: 1/4", L: 180 cm) to the air outlet

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


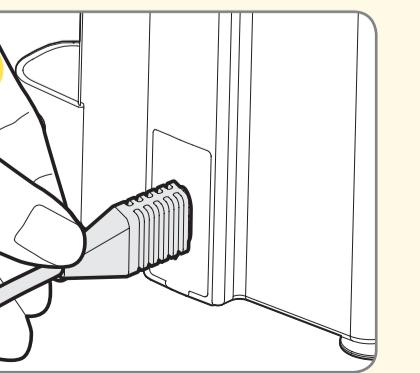
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
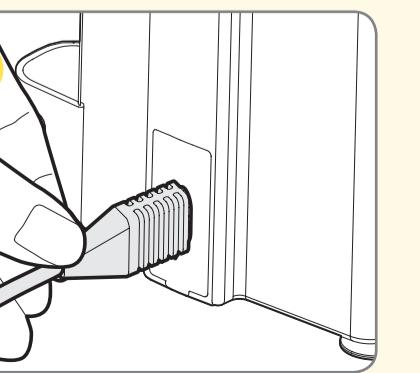
## Product identification
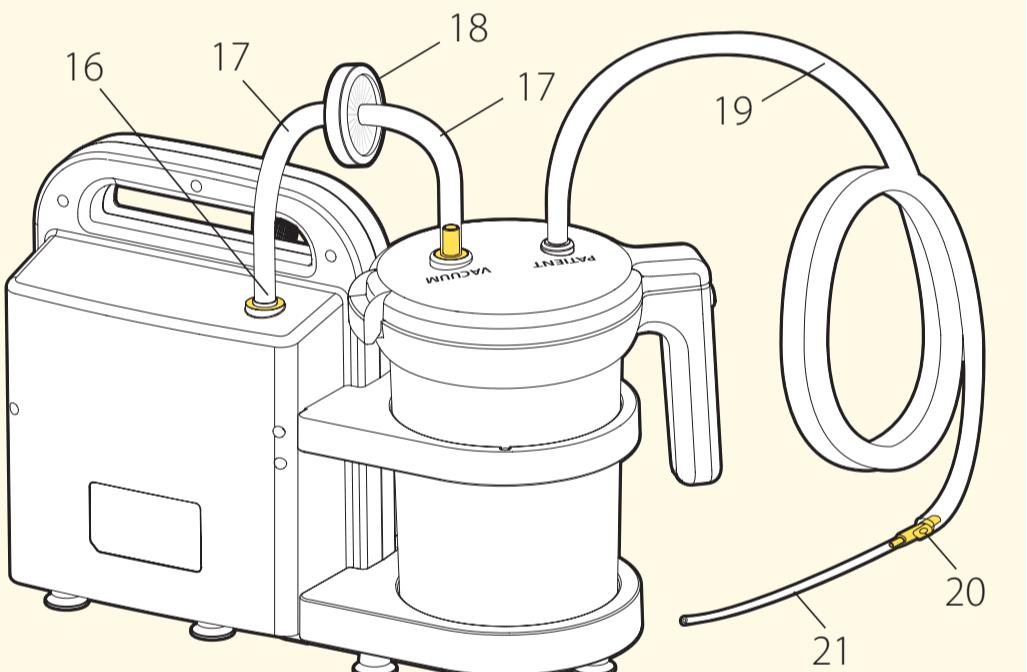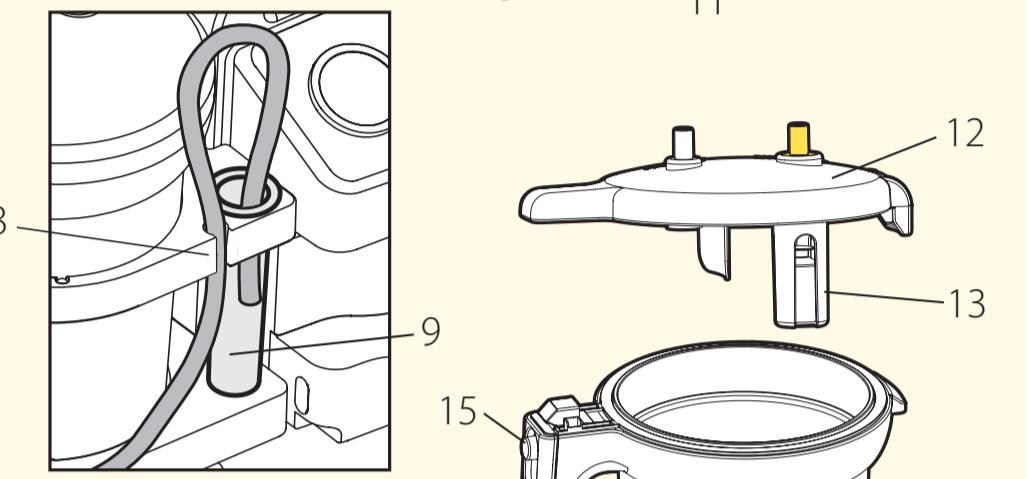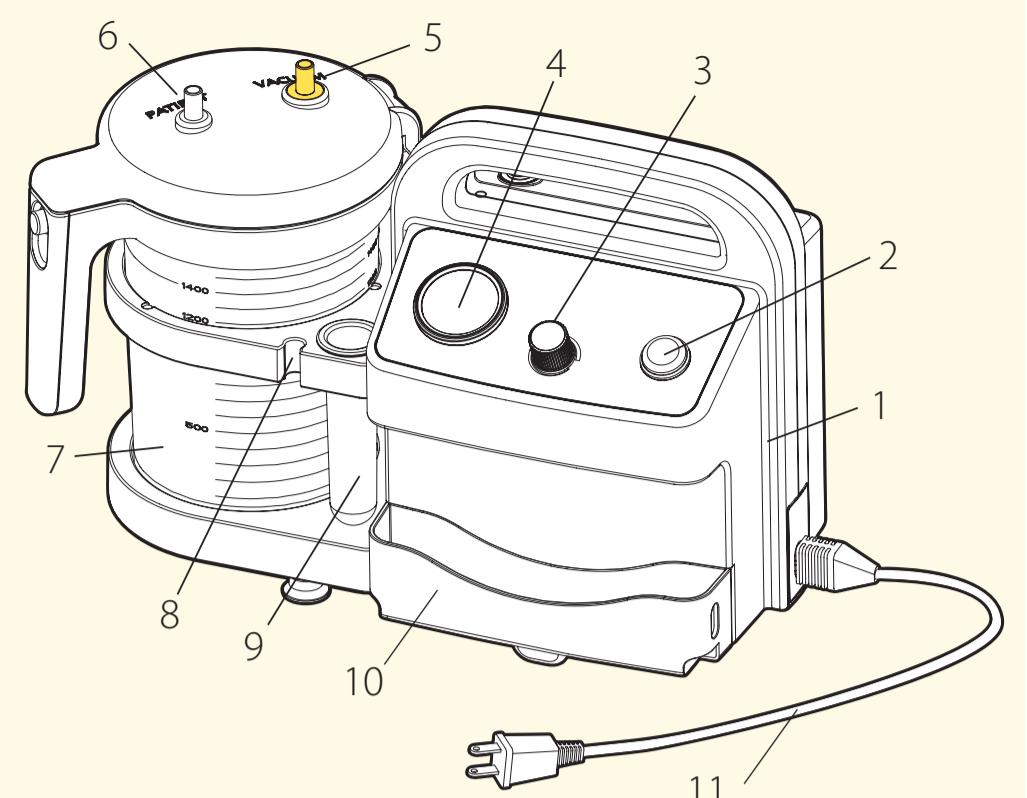
## Assembling Procedures

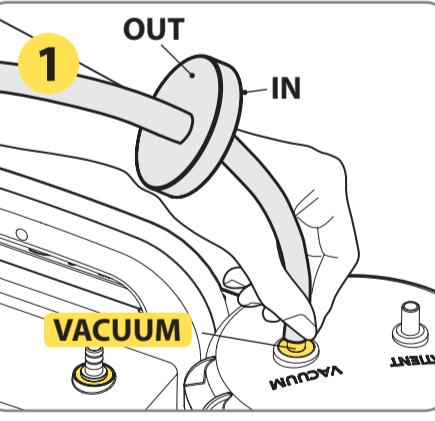**EN** Connect one end of the connecting silicone tubing ( $\phi$ : 8 mm, L: 14cm), with the antibacterial filter marked "IN" side, to the canister VACUUM inlet marked in Yellow

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**RO** Connect the connecting PVC tubing ( $\phi$ : 1/4", L: 180 cm) to the canister "PATIENT" inlet

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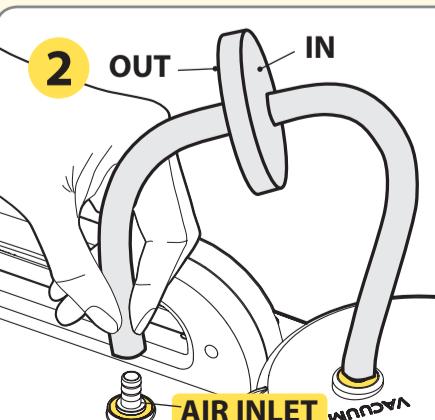
**SK** Pripojit PVC silikonovou hadicku ( $28 \text{ mm}$ ,  $14 \text{ cm}$ )

**EN** Connect the other end, with the antibacterial filter marked "OUT" side, to the "AIR INLET" connector marked in Yellow of the suction unit

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